

Basic Skills Checklist

Fabius GS



A Dräger and Siemens Company

Name: _____

Hospital: _____

Unit: _____

Date: _____



This Skills Checklist is provided for information purposes only, focuses on the Draeger Medical product referenced and related concepts, and is not a substitute for reading the applicable Instructions for Use. The skills checklist assists in training how to use or operate this product. It does not address issues of medical practice or clinical judgement related to the use of this product.

The checklist is intended only as a suggestion for your consideration; it may or may not be consistent with your institutional expectations for staff competency. It is the responsibility of each institution or provider to make their own determinations regarding matters of training and clinical protocol for their employees and the products that they use.

Pass	Needs Review	Skill
		1. Locate Power switch.
		2. Place in STANDBY mode/Take out of sleep mode.
		3. Assemble circuit, gas and suction pressure lines.
		4. Run System test.
		5. Flow sensor calibration.
		6. O ₂ Calibration.
		7. Leak /Compliance Test.
		8. Access alarm logs.
		9. Restore site defaults.
		10. Check scavenger for adequate suction.
		11. Check gas cylinder pressures.
		12. Remove and replace vaporizer.
		13. Fill vaporizer / check vaporizer levels.
		14. Activate machine into Man/Spont.
		15. Adjust Flows/ APL valve.
		16. Set alarms for Man/Spont.
		17. Set ventilation parameters / Begin mechanical ventilation Volume/Pressure/ Pressure Support/ SIMV.
		18. Volume Ventilation - VOL / Pmax / Vt/Freq / TI:TE / TIP:TI / PEEP.
		19. Pressure Ventilation - PCV /Pinsp/ TI:TE / Insp flow / PEEP.
		20. Pressure Support Ventilation.
		21. Adjust alarms for mechanical ventilation.
		22. Turn off mechanical ventilation and return to Man/Spont.
		23. Shut machine down at end of day.