

# Basic Skills Checklist

## Infinity Series Monitors

### Gamma/ Gamma XL

Name: \_\_\_\_\_

Hospital: \_\_\_\_\_

Unit: \_\_\_\_\_

Date: \_\_\_\_\_

Device:  GAMMA     GAMMA XL



This Skills Checklist is provided for information purposes only, focuses on the Draeger Medical product referenced and related concepts, and is not a substitute for reading the applicable Instructions for Use. The skills checklist assists in training how to use or operate this product. It does not address issues of medical practice or clinical judgement related to the use of this product.

The checklist is intended only as a suggestion for your consideration; it may or may not be consistent with your institutional expectations for staff competency. It is the responsibility of each institution or provider to make their own determinations regarding matters of training and clinical protocol for their employees and the products that they use.

Pass	Needs Review	Skill
		1. Unlock and dock the monitor for transport.
		2. Admit patient name and data.
		3. Turn Pacer Detection ON or OFF
		4. Change displayed lead from Lead II to Lead III.
		5. Change NIBP interval time to 30 minutes.
		6. Activate and state the function of All Alarms Off fixed key.
		7. Identify the three alarm grades.
		8. Set SPO <sub>2</sub> low alarm limit to 88%.
		9. Change PVC alarm limit to 20 b per minute.
		10. Change HR archive to STORE.
		11. Access the TREND TABLE.
		12. Access, review and record a STORED EVENT.
		13. Initiate - Strip Recording
		14. Change invasive pressure label from ART to CVP.
		15. Zero the pressure Line.
		16. Activate STANDBY mode, exit STANDBY, resume monitoring.
		17. Discharge monitor.