

# Basic Skills Checklist


## Infinity Multiview/ICS Workstation

Name: \_\_\_\_\_

Hospital: \_\_\_\_\_

Unit: \_\_\_\_\_

Date: \_\_\_\_\_

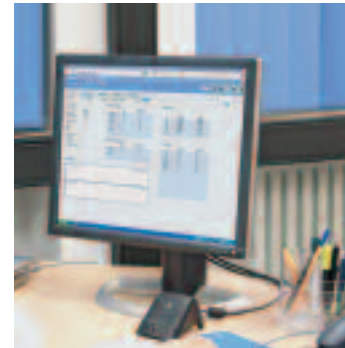

This Skills Checklist is provided for information purposes only, focuses on the Draeger Medical product referenced and related concepts, and is not a substitute for reading the applicable Instructions for Use. The skills checklist assists in training how to use or operate this product. It does not address issues of medical practice or clinical judgement related to the use of this product.

The checklist is intended only as a suggestion for your consideration; it may or may not be consistent with your institutional expectations for staff competency. It is the responsibility of each institution or provider to make their own determinations regarding matters of training and clinical protocol for their employees and the products that they use.

Pass	Needs Review	Skill
		1. Admit patient name and data.
		2. Identify and state the information displayed in a parameter box.
		3. Identify and state the information displayed in the waveform box.
		4. Access and utilize BED SILENCE to silence a patient alarm.
		5. Silence Alarms at MVWS/ICS.
		6. Identify screen display location for a patient specific Alarm/Error message.
		7. Identify the three alarm grades.
		8. Change HR low alarm limit to 45 bpm.
		9. Change archive to STORE for the Arrhythmia RUN.
		10. Initiate printing - Strip Recording.
		11. Initiate a Strip Recording for all monitored patients.
		12. Access TREND TABLE.
		13. Access, review and print a STORED EVENT.
		14. Access Full Disclosure and do caliper measurements.
		15. Access Full Disclosure and set-up to measure with CALIPERS.