

# Basic Skills Checklist

Symphony



A Dräger and Siemens Company

Name: \_\_\_\_\_

Hospital: \_\_\_\_\_

Unit: \_\_\_\_\_

Date: \_\_\_\_\_



This Skills Checklist is provided for information purposes only, focuses on the Draeger Medical product referenced and related concepts, and is not a substitute for reading the applicable Instructions for Use. The skills checklist assists in training how to use or operate this product. It does not address issues of medical practice or clinical judgement related to the use of this product.

The checklist is intended only as a suggestion for your consideration; it may or may not be consistent with your institutional expectations for staff competency. It is the responsibility of each institution or provider to make their own determinations regarding matters of training and clinical protocol for their employees and the products that they use.

| Pass | Needs Review | Skill                                                                                 |
|------|--------------|---------------------------------------------------------------------------------------|
|      |              | 1. Logon to Symphony.                                                                 |
|      |              | 2. Navigate CENSUS SCREEN and select a patient.                                       |
|      |              | 3. Identify and state information displayed in patient area. Access more information. |
|      |              | 4. Change duration of STATUS PAGE.                                                    |
|      |              | 5. Choose etCO <sub>2</sub> waveform.                                                 |
|      |              | 6. Print Report.                                                                      |
|      |              | 7. Access WAVES screen.                                                               |
|      |              | 8. Change the channels to 4.                                                          |
|      |              | 9. Use >< arrows to navigate time.                                                    |
|      |              | 10. Use up/down arrows to change displayed waveforms.                                 |
|      |              | 11. Navigate with Trend view navigator.                                               |
|      |              | 12. Access the EVENTS SCREEN and print a report.                                      |
|      |              | 13. Access ECG Screen.                                                                |
|      |              | 14. Format ECG screen.                                                                |
|      |              | 15. Print ECG Report.                                                                 |
|      |              | 16. Access Trends: change Interval and duration.                                      |
|      |              | 17. Access and explain BED VIEW.                                                      |
|      |              | 18. Identify and explain Buttons on top right of display screen.                      |
|      |              | 19. Logout.                                                                           |