



QUICK REFERENCE

Infinity Series Monitor

Delta/Delta XL

Advanced Parameter Pods

Warning! This "Quick Reference" is not intended to be or, nor is it, a replacement for the instructions for use. For full information concerning the performance characteristics of the Dräger Medical devices described in the Quick Reference, each user must carefully read and fully comprehend the Instructions for Use before operating the respective device.

MultiMed 12-Lead ECG

- Add the additional 4 or 5 precordial leads
- Connect the MultiMed-12 Pod to auxiliary port on the side of the monitor
- Connect the patient's existing 5 or 6 lead ECG set to the MultiMed-12 Pod
- Disconnect the duplicate MultiMed pod from the monitor

Setup ECG Report

- Highlight Heart Rate parameter box, click to confirm
- Choose ECG Options, click
- Choose ECG Report Setup click to confirm
- Enter all appropriate data

Print 12 -Lead

- Press Fast Access
- Choose Reports, click to confirm
- Choose ECG Report, click to confirm

Note: 12 leads of ECG rhythm may be printed from a modular monitor (with Aries option) to the laser printer without a MVWS; however, to print the diagnostic quality 12-Lead in a preconfigured format the MVWS with Rest ECG Option is required.

etCO₂ POD OR MODULE

Mainstream or Sidestream

- Check correct mode – mainstream/sidestream is chosen in the parameter box menu.

Note: Neonatal Sidestream etCO₂ monitoring is NOT supported using this pod.

Connection

- Connect the etCO₂ pod cable to the round auxiliary port on the side of the monitor or slide the module into the slot on the back of the Delta or Delta XL.
- Connect the etCO₂ Capnostat cable to the pod or module. (Note: There is a special nasal cannula and sample tubing to perform sidestream measurement)
- The pod will begin a warm-up process lasting 2-3 minutes. Watch the message area of the monitor screen for instructions to initiate the zero and reference calibration process.

QUICK REFERENCE

Infinity Series Monitor

Delta/Delta XL

Advanced Parameter Pods

etCO₂ POD OR MODULE, continued

Zero and Reference the Capnostat Sensor

- Slide the Capnostat onto the "0" window until it clicks into place (line up arrow).
- Watch for "0" in the parameter box.
- At the prompt, place the Capnostat sensor on the reference window. 38+/-1 will display in the parameter box
- The message "Zero and Reference have been completed" will display with a prompt to calibrate the airway adapter.

Calibrate the Airway Adapter

- Slide the Capnostat sensor onto the airway adapter until it clicks (line up arrow).
- Touch the **etCO₂ parameter** box.
- Touch **Adapter Cal** (make the adapter is in room air).
- Click **Continue**.
- Watch the message area for configuration that calibrated is complete.
- Place the airway adapter and Capnostat sensor inline proximal to the 'Y' of the ventilator circuit. Ensure the sensor is positioned to the top part of the airway adapter and tubing (like a bridge over the tubing).
- etCO₂ waveform and parameter measurements will display.

Connection and Setup

- Attach the connecting cable from the pod to the pod com port on the left side of the monitor
- Attach the appropriate sample line to the pod
- Highlight etCO₂ Parameter Box to access and change:
 - O₂ compensation
 - Scale
 - Sweep Speed
 - Averaging
 - Measurement Mode
 - Alarms

The Microstream pod does not require calibration