



## QUICK REFERENCE

### Infinity Series Monitor Gamma XXL

*Warning! This "Quick Reference" is not intended to be or, nor is it, a replacement for the instructions for use. For full information concerning the performance characteristics of the Dräger Medical devices described in the Quick Reference, each user must carefully read and fully comprehend the Instructions for Use before operating the respective device.*

#### Getting Started

The rotary knob at the bottom right corner of the monitor rotates left and right. It will highlight selections and is pressed (CLICK) to confirm the selection.

#### Connecting a patient

- Prep the patient's skin, place ECG electrodes appropriately
- Place SpO<sub>2</sub> sensor on patient
- Place blood pressure cuff on patient
- Press **Main Screen** key to begin active patient monitoring

#### Patient admit

- Admission via MWWS is faster, however, you may admit from the bedside monitor
- Press **Menu** key
- Choose **Admit/Discharge**, press rotary knob to confirm.
- Choose **Admit** and enter required patient data.

#### Alarm Levels

- **L-T (Life threatening)** alarm flashes red and sounds two alternating tones. L-T alarms must be manually silenced. Press the **Alarm Silence** key or suspend using the All Alarms Off key
- **SER (Serious)** alarm flashes yellow and sounds the same tone twice. A serious alarm will reset if the causing the alarm no longer exists.
- **ADV (Advisory)** alarm flashes white and sounds a single tone. Alarm resets if the violation ceases.

#### Understanding Fixed Keys

Alarm Silence	Silences the triggered bedside alarm for 60 seconds
Alarm Limits	Allows limits to be set on all parameters
All Alarms Off	Initiates 2 min. suspension of all alarms
Record	Sends timed recordings to R50
NBP Start/Stop	Starts single NBP measurement Starts interval mode, if preset Stops measurement
Fast Access	Fast access to frequently used functions
Main Screen	Returns to main monitor screen, removing any menus
Menu	Accesses all monitor functions

# QUICK REFERENCE

## Infinity Series Monitor

### Gamma XXL



#### Accessing Parameter Box Functions

All parameter box functions are accessed using the rotary knob to highlight the desired parameter, then press the rotary knob (click) to confirm selection.

#### ECG

##### Change Displayed ECG Leads

- Highlight **Heart Rate** parameter box, click to confirm
- Choose **Lead Setup**, click to confirm
- Choose **Channel 1**, click to confirm
- Select desired lead, click to confirm
- Repeat steps to change other channels

##### Activate Pacemaker Detection

- Highlight **Heart Rate** parameter box, click to confirm
- Choose **ECG options**, click to confirm
- Choose **Pacer Detection ON**, click to confirm

##### Arrhythmia

- Highlight Heart Rate parameter box, click to confirm
- Highlight ARR parameter box, click to confirm
- Customize type of alarm, rate, count and archive function as desired.

#### Accessing Parameter Box Functions

All parameter box functions are accessed using the rotary knob to highlight the desired parameter, then press the rotary knob (click) to confirm selection.

#### ECG

##### Change Displayed ECG Leads

- Highlight **Heart Rate** parameter box, click to confirm
- Choose **Lead Setup**, click to confirm
- Choose **Channel 1**, click to confirm
- Select desired lead, click to confirm
- Repeat steps to change other channels

##### Activate Pacemaker Detection

- Highlight **Heart Rate** parameter box, click to confirm
- Choose **ECG options**, click to confirm
- Choose **Pacer Detection ON**, click to confirm

##### Arrhythmia

- Highlight **Heart Rate** parameter box, click to confirm
- Highlight **ARR** parameter box, click to confirm
- Customize type of alarm, rate, count and archive function as desired.



## NBP

- Highlight **NBP** click to confirm
- Choose **Interval Time**, click to confirm
- Dial in the desired time interval, click to confirm
- Press **NBP Start/Stop** key to begin the interval measurements. A bar appears under the NBP showing the interval time.
- To stop a measurement press **NBP Start/Stop** key.

## Respiratory Monitoring

- Highlight **Respirations** parameter box, click to confirm
- Enter desired menu options

## SpO<sub>2</sub> MONITORING

### Change Waveform Size

- Highlight **SpO<sub>2</sub>** parameter box, click to confirm
- Adjust waveform size appropriately, click to confirm

### Temperature Monitoring

- Highlight **TEMP** parameter box, click to confirm
- Enter desired menu options

### Alarm Limits

- Press **Alarm Limits** key
- Choose the limit(s) to be changed, click to confirm and dial to the desired value, click to confirm
- Verify the alarm is ON/OFF as desired

### Recordings

- Press **Record** key for a timed R50 strip recording
- For a continuous recording to the R50, press **Record** key, choose **Continuous Record**, click to confirm.

## Access Saved Data

### Trends

- Press **Fast Access** key
- Choose **Trend Graph** or Trend Tables, click to confirm
- Select desired time interval, click to confirm
- Choose **Report**, it will print to the laser printer

### Review Stored Events

- Press **Fast Access** key
- Choose **Event Recall**
- Press record key to send to R50N

# QUICK REFERENCE

## Infinity Series Monitor

### Gamma XXL



#### Menu Functions

##### Change Patient Category

- Press **Menu** key
- Choose **Admit/Discharge**, and click to confirm
- Choose **Admit**
- Select desired category, click to confirm

*\*\*Patient category is displayed on screen\*\**

##### Place Monitor in Standby

- Press **Menu** key.
- Choose **Standby**, click to confirm

##### Discharge

- Press **Menu**
- Choose **Admit/ Discharge**, click to confirm
- Choose **Discharge**, click to confirm
- Choose **Discharge** to confirm discharge

*Note: The monitor does not need to be powered off between patients.*

#### Invasive Pressures

##### Connection & Setup

- Connect Y-cable to monitor
- Connect transducers directly to monitor or via intermediary cable as necessary

##### Zero Transducers

- Turn stopcocks off to patient and open to air level with the phlebostatic axis (except ICP monitoring)
- Highlight the **Pressure** parameter box, click to confirm
- Choose **Zero** click to confirm

##### Change Labels

- Highlight the **Pressure box**, click to confirm
- Choose **Labels**, click to confirm
- Highlight the label you wish to change, click to confirm, choose the desired label, click again to confirm.

*This Quick Reference is intended as a supplemental learning tool and does not replace the Operating Instructions supplied with the equipment.*